

PAIRING THE WIRELESS FOOT CONTROL WITH THE CONTROL BOX

1. OPEN BATTERY COMPARTMENT


Open the battery compartment on the underside of the pedal.
Turn the wireless foot control over and open the compartment.
The cover may be secured with screws.

2. INSERT BATTERIES

Insert the batteries according to the plus and minus markings.
Ensure the batteries are securely placed.

3. STAND NEAR THE CONTROL BOX

Hold the foot control at a maximum distance of two metres from the control box of the treatment bed.

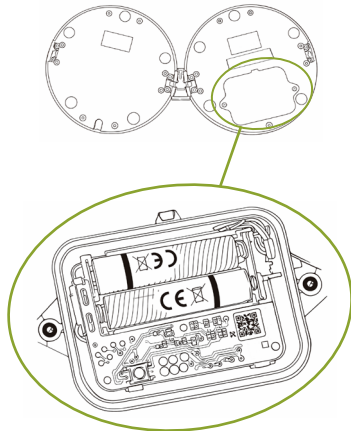

4. ACTIVATE PAIRING MODE

Press the button underneath the foot control for approximately three seconds.
You will hear a beep, and the indicator light will turn on. Pairing mode is now active.

5. CONTROL BOX

Move the foot control closer to the control box. Bring it within 10 cm of the control box.
The beep will change from slow to fast.

6. CONFIRM PAIRING

Press the button underneath the foot control again to confirm pairing.
You will hear two short beeps, indicating successful pairing.